INSTRUCTION MANUAL

2-Phase Electronic Breast Pump LOVI - Prolactis

Professional protection of breastfeeding

Please read this instruction manual carefully before using the device.

Important safety information:

Failure to observe this instruction manual and the safety information may result in danger and the device failure. Use the device only as intended and described in this manual. The information contained in this manual is subject to appropriate technical changes.

INTRODUCTION

Congratulations on your purchase of the 2-Phase Electronic Breast Pump LOVI! Prolactis Breast Pump is a professional aid to support breastfeeding and solve problems of lactation. Ideal for situations requiring frequent pumping. It also provides support for active and working mums who require alternative ways to feed breast milk. Enables painless, comfortable milk expression in a short time. Tested in cooperation with Polish hospitals*.

Prolactis Breast Pump is equipped with two expression cycles inspired by baby's sucking rhythms: stimulation cycle and expression cycle.

Stimulation cycle – imitates fast suckling moves that stimulate the production and milk let down.

Expression cycle – reflects slower and deeper effective suckling. When milk start to flow, the baby must put more effort into suckling. Milk slowly becomes more dense and filling, that is why the suckling rhythm and the pace slow down.

Range of rhythm and power of Prolactis Breast Pump suction has been developed to reflect the natural suckling process and how the rhythm of sucking changes during breastfeeding.

Using both cycles, you will express more milk in less time.

Additional diaphragm and handle, included in the Prolactis set, enables manual milk expression. Converting only few parts, Electronic Breast Pump LOVI Prolactis can be transformed into Manual Breast Pump LOVI Protect.

Breast milk is the healthiest and most valuable food for newborns and infants. It is recommended as a sole food given to your child in the first 6 months of life. After six months, it should be combined with solid food.

NOTE! Do not use a breast pump in circumstances when a hungry baby fails to empty the breast itself, i.e. if there is any possibility that your milk ducts are blocked.

USEFUL HINTS

When to use a breast pump?

If possible, wait until lactation stabilizes, which usually occurs within 2-4 weeks after birth.

Exceptions to this rule:

- If your breasts produce too much milk, they become swollen and hard (engorged). Expressing a small amount of milk before or during a feed can give you relief and help your baby to grasp the nipple.
- If you have inverted nipples (which may occur because of long intervals between feeds.)
- If you have a concave, flat or damaged nipples (using a breast pump can improve nipple shape).
- If you feel overflow between breast feeds and do not want to wake the baby.
- The baby is sick and does not want to suckle.
- If your breasts produce too little milk, using the pump between feeds will help increase milk production.
- When you are working or away from home and want to continue breastfeeding, you should regularly express milk using a breast pump to sustain the natural process of lactation.

* Study LOVI Prolactis breast pumps and LOVI Protect the group of 40 women, supervised by midwives and lactation consultants.

How often should you express milk?

If you temporarily cannot breastfeed it is recommended that you express milk using Breast Pump at least 8 times a day. This will keep milk production at the appropriate level. In the beginning you may experience difficulty expressing, but after a few days you will be pumping more. Collecting one portion of milk may require several sessions.

BASIC PRODUCT INFORMATION

Breast Pump PA203S

2-Phase Electronic Breast Pump LOVI – Prolactis, (later referred to in the manual as, a Breast Pump'), is a product designed for breastfeeding mothers in order to express breast milk. It enables collection of milk for later usage i.e. in the case of separation from the child. The Breast Pump helps maintain lactation and prepares for your planned absence or random events such as power outages, treatment, cracked and sore nipples.

- The device is intended for personal use.
- The device warranty is 24 months.
- Before using, read the instruction manual and follow the rules of safe use.
- Keep the manual together with the device.
- The expected life of the product is 5 years. The life of silicone parts is 2 years. Silicone parts may change color permanently. These elements must be replaced after finding discoloration or after the estimated time of use.

Follow the recommendations on the correct use of the device and its accessories. Never use a damaged or incomplete / broken device. Damaged or worn parts should be replaced immediately. The manufacturer is not liable for damages resulting from the misuse of the device, the resulting damage in this way cannot be the basis for the claim or warranty. Making any repairs or modifications to the product by unauthorized persons, results in loss of warranty and may result in risks to health and life.

SAFETY RULES

- If you experience problems while breastfeeding, or feeding causes pain, it is necessary to seek medical advice, before using the Breast Pump.
- The first use of the Breast Pump is best done under the supervision of specialists.
- It is recommended to read specialist literature and get to know the principles of baby feeding.

NOTE! Insufficient suction may result in a smaller amount of milk, but too high suction can cause soreness and reduce production of milk. If you notice this type of failure, you should adjust the suction power (see SUCTION LEVEL).

- Keep the product clean. Prevent deposition of dust and other contaminants that could adversely affect the performance of the product.
- Attention to hygiene is important for the safety of mother and child.
- Some elements of the device are not designed to be cleaned, protect them against water and moisture.
- Connect the unit to an efficient and secure mains.
- When using the product outdoors, only using battery power is permitted.

Correct disposal of this product (waste electrical & electronic equipment). Applicable in the European Union and other European countries with separate collection systems. This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

Manufacturer declares under our sole responsibility that the product typed In this manual fulfils the Essentials requirements of medical devices directive 93/42 EEC + 2007/47/EC for class II a.

The product is made in the second class protection.

The product is not a toy. Keep all parts away from children. Retain the instruction for later use. Batch code and expiry date placed on the packaging.

PRODUCT MISUSE

- Never use a defective product, with visible damage characteristics.
- It is forbidden to use the device while bathing.
- Never use the product in locations exposed to contact with water, for example near a bathtub, wash bowl, kitchen sink or shower enclosure.
- It is forbidden to use the device if it is possible to fall asleep.
- Do not connect the device outdoors to the AC adapter.
- Never use a product that has been dropped or has fallen on a hard surface. In this case, the product must be returned to service in order to be checked.
- Never use the product discordant with the purpose and conditions of use.

TECHNICAL DESCRIPTION

Battery operation	4 x AA 1,5 V – alkaline
Power operation	230 V / 50 Hz / 0,4A max; output: =12V / 1A
Maximum power	12 W
Weight (with AC adaptor without batteries)	516 g

HOUSING

WARNING!

1. Make sure the polarity (+) and (-) is correct as indicated.
2. Remove batteries if pump is not in use for a long period of time.
3. Dispose of batteries according to regulations for environmental protection and recycling.
4. Replace batteries as a simultaneous set.
5. Remove battery from device when not in use for more than 24 hours.
6. Do not tamper with or attempt to repair the device.

SET COMPONENTS

Fig. 1 – PROLACTIS SET PARTS

1. Electric Pump main unit
2. AC adapter
3. Vacuum tube with adaptors
4. Expression collection combo kit
5. Food storage kit
 - a. Screw cap
 - b. Soft sealing disc
 - c. Hood/cup with scale
6. Breast Pump LOVI Protect Parts (for manual pumping)
 - a. Adjusting valve stem
 - b. Diaphragm
 - c. Handle
 - d. Diaphragm cover
7. Spare parts
 - a. One-way valve - 2 pcs
 - b. O-ring - 2 pcs
 - c. Let-down silicone cushion – 1 pc
 - d. Diaphragm (for electric pump) – 1 pc
8. Prolactis storing case
9. Disposable Breast Pads LOVI 'Day&Night' – 10 pcs

Fig. 2 – EXPRESSION COLLECTION COMBO KIT

1. Cap
2. Diaphragm
3. Cup with o-ring
4. Let-down silicone cushion
5. Breastshield body
6. One-way valve
7. Bottle adapter
8. Bottle

Fig. 3 – ELECTRIC PUMP MAIN UNIT

1. LCD screen
2. Vacuum adjusting wheel
3. Let-down button
4. M button
5. On/Off button & power LED
6. Battery cover
7. Tubing adapter inlet
8. AC adapter inlet

Fig. 4 – LCD SCREEN GUIDE

1. Suction level indicator
2. Stimulation cycle indicator
3. Expression cycle indicator
4. Time indicator
5. Power indicator
6. M icon (memory)
7. Alarm



HOW TO ASSEMBLE THE BREAST PUMP

Hold the „horn” of the breastshield body in one hand. Make sure the bottle-thread side opening is facing up.

With the thumb and fore-finger of the other hand, hold the tip of the one-way valve and place the flat circular bottom into the opening. Be sure not to stretch, bend, or twist the tip of the one-way valve. Seal all the edges firmly. With your thumb and fore-finger, gently seal any openings on the tip of the one-way valve and shape into a straight line.


Note: The one-way valve tip does not have to align with anything, as long as all the edges are securely sealed. **This is critical to the optimal performance of the pump.**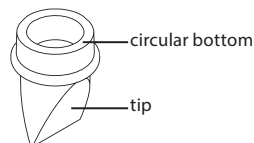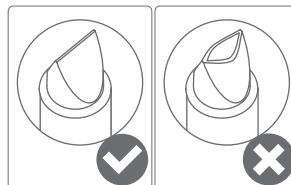


Carefully twist the bottle adapter to the breastshield body for a secure fit.

Note: Be sure not to bump the one-way valve when twisting the breastshield body & the bottle adapter pieces together. If you bump the one-way valve, repeat above steps.

With the bottle adapter in one hand, twist the collection bottle for a secure fit.

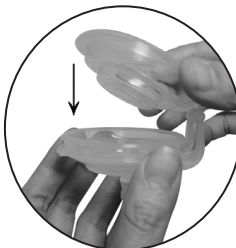
With the cup in one hand, place the silicone diaphragm in the cup (matching the shape of the cup). Press around the edges of the diaphragm for a secure fit. Make sure the diaphragm is evenly mounted on the cup and there is no gap in between them.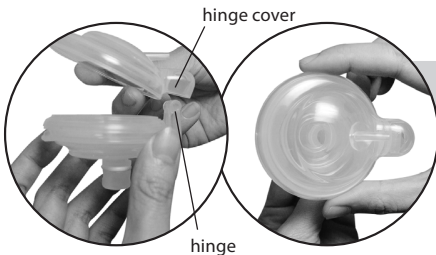

Place the cap on the cup & diaphragm. (Be sure to align the hinge and hinge cover). Press tightly to seal.

Twist the electronic pump diaphragm kit clockwise into the breastshield body until it locks.

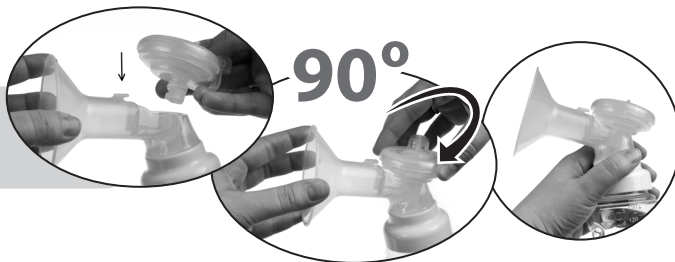With the breastshield body in one hand, place the silicone cushion in the breastshield body matching the shape of the flange. Secure the lip of the cushion around the edges of the flange.

Note: Your Electronic Breast Pump Prolactis will work with or without the silicone cushion. Your comfort depends on your preference.

POWER OPTIONS

Electronic Breast Pump Prolactis may be operated under 2 following power supply modes

Using your AC adapter (included):


Attach the power cord to the electric pump base. Plug the AC adapter into an electrical outlet.

Electrical safety can only be achieved when using the AC adaptor which was provided with the pump. Use of any another adaptors causes concern for the safety of the product and/or user and voids the warranty.

It is permitted to operate using AC adaptor without removing the battery from the container.

NOTE! If there are no working batteries in the pump and the devise is unplugged from the electrical outlet, all previous settings are erased from the memory.


NOTE! AC adapter pins may vary depending on the country of destination.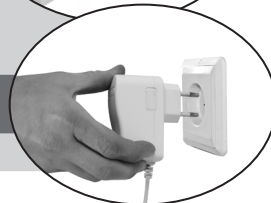

Using 4 AA Alkaline batteries (1,5 V)


WARNING!

- Always replace all 4 batteries at the same time.
- Remove all batteries from the electric pump base if the Electronic Breast Pump is not going to be used for extended periods of time.
- Properly dispose of all batteries, according to regulations for environmental protection and recycling.
- Alkaline batteries may present a risk of fire or chemical burn if mistreated. Do NOT disassemble any battery. Do NOT place batteries near heat source or fire. Dispose of used batteries properly. Do NOT dispose of batteries in fire. Keep away from children.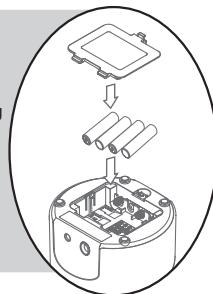


Open the battery cover.

Remove the old batteries if you are replacing them.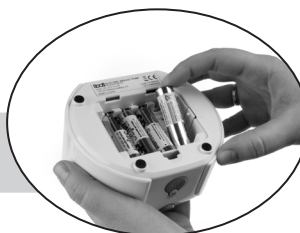

Insert 4 new AA alkaline batteries.

Note: Make sure the polarity is correct as indicated.

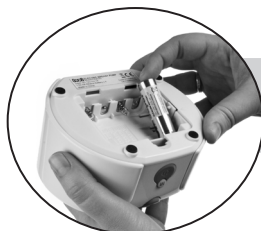Close the battery cover, making sure it „snaps“ into place.

PUMPING INSTRUCTIONS


Before each use:

- Wash your hands thoroughly before using the pump.
- Ensure that the pump is pre-washed and dried, if not, follow the instructions located below.
- Ensure that there is no liquid or moisture in the silicone tube. Never operate the pump if you notice any water in the silicone tube.

Note! Do not mouth blow the silicone tube in order to remove water.

Take the white end of vacuum tube and connect to the tubing connector located on the cap. Push together firmly for a tight seal.

Note! Ensure that there is no liquid or moisture in the silicone tube.

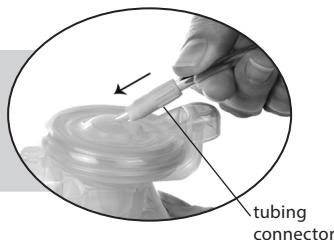tubing
connector

Make sure the tubing connector is properly inserted into the electric pump base.
Do not remove the film. Lift tab to insert the tubing adaptor. Pull the tab down to cover the inlet after use to keep the motor dust free.

Hold the expression collection combo kit to your breast so that your nipple(s) are centered inside the breast shield.

IMPORTANT NOTE! Make sure that funnel or silicone cushion (if you use it) clings tightly to the breast, otherwise the suction strength may be reduced.
If you want to increase the suction strength, use the funnel without the silicone cushion.

Press the On/Off button  to begin pumping.

On/Off

The LCD screen shows the time of your last pumping session for a few seconds, the pump will automatically begin in the stimulation cycle.

Adjust the suction in the stimulation cycle to your comfort level while pumping by rolling the vacuum adjusting wheel  up (+) or down (-).

vacuum adjusting wheel

let-down button


If you prefer to bypass the pre-set stimulation cycle  or your milk lets down sooner than 2 minutes, press the let-down button  to switch to the expression cycle. 

stimulation cycle

expression cycle

After two minutes, the stimulation cycle will end and automatically switch to the expression cycle.

The expression icon  will appear and indicate that you are in the expression cycle.

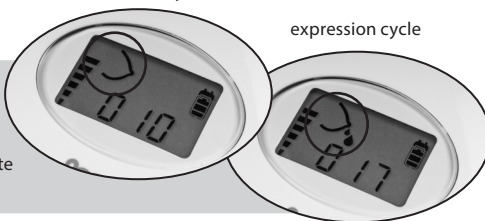vacuum adjusting wheel

Adjust the suction in the expression cycle to the most comfortable and effective level while pumping by rolling the vacuum adjusting wheel  up (+) or down (-).

When your pumping session is finished, turn the pump off by pressing the On/Off button. 

On/Off

After the pump has been turned off, the display will remain on for a few seconds. The display will show the total amount of pumping time, then automatically switch to standby/clock mode, or alarm clock mode if the alarm was turned on.

Note: Make sure there is no suction on the nipple(s) before removing the breast shield. If necessary, break the suction by inserting a finger between the breast shield and the breast, similar to the way you break away from your baby's suction.

Note: If the Electronic Breast Pump Prolactis is left on, but unattended for more than 30 minutes, the pump will automatically shut-off.

SUCTION LEVEL

Whenever you select a suction level (in both stimulation & expression cycle), your Electronic Breast Pump Prolactis remembers until you change it next time. This feature becomes useful after you are familiar with this pump. Your Electronic Breast Pump Prolactis keeps your previous pumping level without having to reset your preferred level each time you pump.

To set your preferred suction level:

1. In stimulation cycle, adjust the stimulation suction to the level you find most comfortable by rolling the vacuum adjusting wheel  up (+) or down (-).
2. In expression cycle, adjust the expression suction level to the highest effective, yet still comfortable level by rolling the vacuum adjusting wheel  up (+) or down (-).

If food gets under the silicone massaging cushion, reduce the suction level so that milk keep up flowing into the bottle.

NOTE: If you find the suction power insufficient (even at the highest setting) try pumping without massage cushion.

M BUTTON (MY FAVORITE MODE)

One of the great features of your Electronic Breast Pump Prolactis is the M button (My Favorite Mode). You can easily set your personal let-down time by press the M button.

To set your personal let-down time:

1. After few times of pumping, you will find your let-down time about the same in every pumping session.
2. Start pumping as you normally do. Your pump starts in stimulation cycle with your last preferred stimulation level.
3. Press the let-down button  when your milk starts to flow to begin the expression cycle with your last preferred expression level.
4. Press and hold the M button for three seconds, until the M icon flashes . Your personal let-down time has been set.
5. Continue expressing and finish your pumping cycle.

To pump with your personal let-down time:

1. Turn the pump on.
2. Press the M button. (Do NOT hold more than 3 seconds)
3. Your pump automatically switches from stimulation cycle to expression cycle at your personal let-down time.

NOTE: You CANNOT adjust the suction level or switch between stimulation cycle and expression cycle while in custom mode. If you have to, stop My Favorite Mode by pressing the M button.


SETTING THE TIME

How to adjust the clock:

1. While in the clock/standard by mode, press and hold the M button  for 3 seconds to go into clock setting mode. A beep will sound and the AM/PM icon will flash.
2. To change between AM/PM, use the vacuum adjustment wheel  up (+) or down (-).
3. Press the let-down button  once to change from AM/PM to the hour. The hour number will flash.
4. To change the hour, use the vacuum adjustment wheel  to increase (+) or decrease (-) the hour.
5. Press the let-down button  again to change the minutes. The minute number will flash.
6. To change the minutes, use the vacuum adjustment wheel  to increase (+) or decrease (-) the minutes.
7. Press the let-down button  one last time to save your settings.

NOTE: While in any clock setting mode, if no adjustments are made within 10 seconds, the setting will automatically save and the pump will return to clock/standby mode.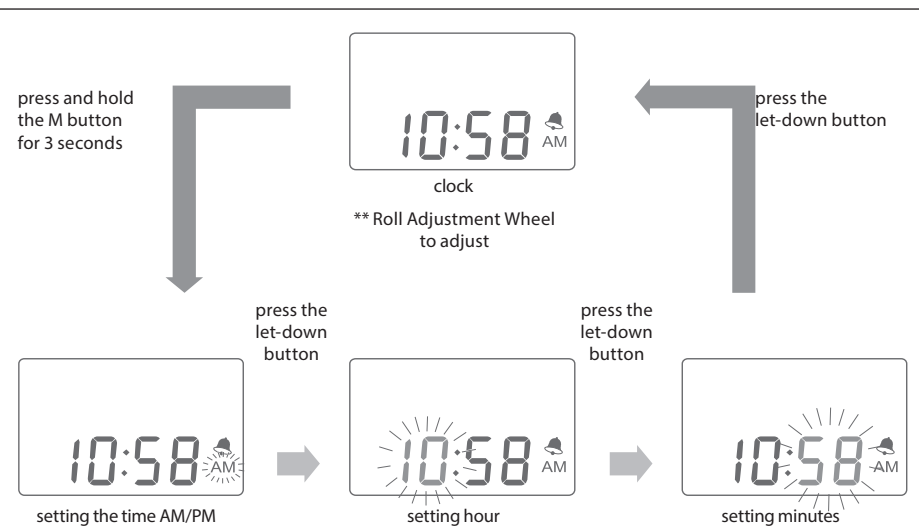

ALARM CLOCK MODE

It is advised to pump regularly in order to maintain a good milk supply from your body. The alarm clock mode is designed to maintain a regular expression cycle, by reminding you when it's time for your next pumping cycle. After setting the time interval between each pumping, the alarm countdown will automatically start after the previous pumping cycle has stopped. An alarm will sound when the pre-set time has passed.


How to Adjust the Alarm:

1. In clock/standby mode, press the M button  once to go into alarm adjusting mode.
2. Two sets of time will display alternately, the alarm interval time (the time between each expression cycle) and the next alarm time.
3. Roll the vacuum adjustment wheel  up (+) or down (-) to adjust the alarm interval time. The adjustment increases (+) or decreases (-) in 15 minute increments. The next alarm time will be adjusted automatically according to the last time the pump was used.

NOTE: The alarm interval time is the time from the beginning of one expression cycle to the next expression cycle. It is not the time from the ending of one expression cycle to the next one.

4. Press the let-down button  to switch to the alarm On/Off setting.
5. Roll the vacuum adjustment wheel  up (+) or down (-) to turn the alarm function On/Off.
6. Press the let-down button  again to save your setting.

NOTE: While in alarm adjusting mode, the setting will be automatically saved and the pump will return to clock/standby mode if no adjustment was made within 10 seconds.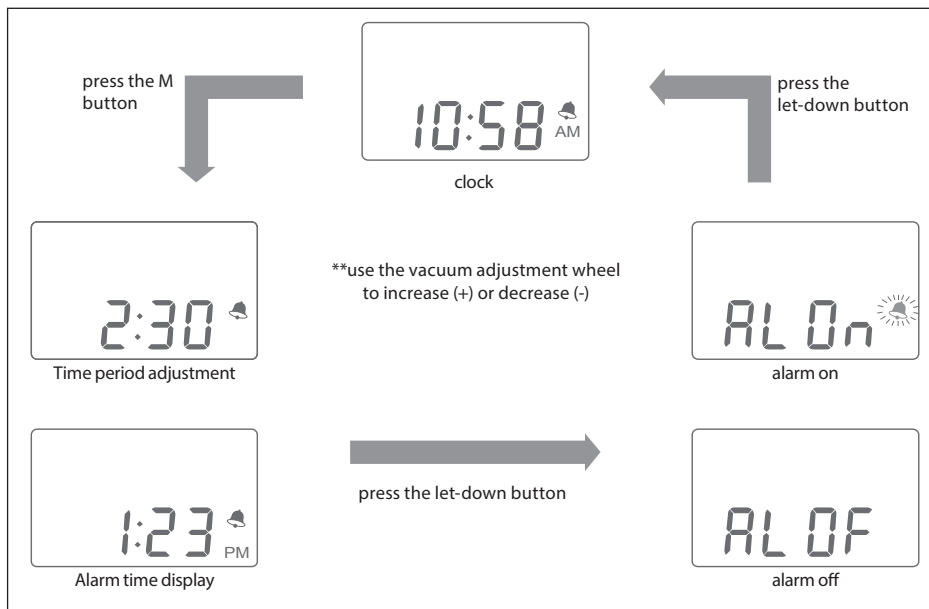

MANUAL PUMPING

Converting into 2-Phase Manual Breast Pump LOVI - Protect

The set includes components for manual pumping. With these Electronic Breast Pump Prolactis easily turns into the 2-Phase Manual Breast Pump LOVI Protect.

To assemble and correctly use the Manual Breast Pump LOVI Protect, please refer to the separate instruction manual included in the packaging.

CLEANING INSTRUCTIONS

Sanitize prior to first use:

Before you use your Prolactis Breast Pump for the first time, disassemble and boil all parts (coming into contact with the breast or food) in water for 5 minutes.

- Be sure to use enough water to keep all parts afloat while boiling.
- **Do NOT boil tubing or tubing adaptor.**
- Remove all parts from the water immediately after boiling.
- Completely air- or towel-dry before reassembly and next use. Dried parts should be stored in an airtight container or a plastic bag.



Partially dry components may affect the operation of your Electronic Breast Pump. Make sure that all parts are completely dry before use.

Cleaning instructions:

- Set aside the tubing and tubing connector. These items do not need to be washed or sanitized because the diaphragm prevents them from coming in contact with the breast milk.
- All parts of the expression collection kit and electronic-pump diaphragm kit may be hand washed in hot water with mild detergent.
- Do NOT use solvents or abrasives.
- Rinse with hot, clean water.
- Suitable for disinfection in electric sterilizers. WARNING! The absence of fluid in the sterilization unit, can cause deformation of the breast pump parts.
- To clean the electric pump base, wipe the base with a damp cloth, using only water or water mixed with a mild soap.
- **DO NOT WASH THE TUBING, THE TUBING ADAPTOR OR ELECTRONIC PARTS.**
- One way valve should be cleaned very carefully. Do not allow the food to dry out in the valve.

NOTE! Always disassemble the valve grasping the circular base with two fingers. Never grasp the end of the valve as it can get damaged and weaken the strength of suction.

Caution: Take care when cleaning the one-way valve. If it gets damaged, your breast pump will not function properly. To clean the one-way valve, rub it gently between your fingers in warm soapy water, then rinse thoroughly in clean warm water. Do not insert any objects into the one-way valve as this may damage the valve.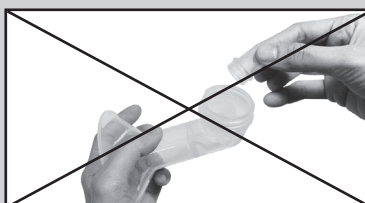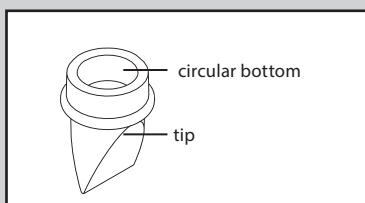

Note: Before each use, visually inspect the individual components for cracks, chips, tears, discoloration or deterioration, as they may affect the efficiency of the breast pump.

TROUBLESHOOTING

While using the breast pump you may encounter small difficulties. Before you decide to place the claim, try to solve these minor problems by yourself. Here are some useful tips that will solve most problems faced.

Interruption of electrical service:

If you experience an interruption of electrical service, due to a power surge or other factors, during normal operation of your Electronic Breast Pump, try the following:

- Unplug the AC adapter cord from the electrical outlet and wait for five seconds.
- Plug the AC adapter cord back into the electrical outlet.
- If problem persists, try a different electrical outlet.

Radio frequency energy:


This equipment generates, uses, and can radiate radio frequency energy. If not installed and used in accordance with the instructions, may cause harmful interference with other devices in the vicinity. Try the following:

- Reorient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a different circuit from that which the other device(s) are connected.
- Consult the manufacturer or a field service technician for help.
- There is no guarantee that interference will not occur.

Decreased suction:

If it feels like the suction has decreased, try the following:

- Detach the tubing and tubing adapter from the back of your Electronic Breast Pump and place a finger over the pump inlet.
 - If you do feel suction, the electric pump base is operating properly.
 - If you do NOT feel suction, try unplugging the AC adapter from the electrical outlet. If problem persists, change electrical outlets.
- Make sure the expression collection combo kit is assembled correctly.
- Inspect the one-way valve for any tears or pinholes. This component is critical to achieving proper suction. If damaged, replace the one-way valve with one of the spare valves that is provided with your Electronic Breast Pump.
- Make sure that the diaphragm is properly set up and there is no gap between the cup with o-ring and the diaphragm.
- Make sure that the o-ring is properly placed on the cup and correctly assembled with the breastshield body.
- Make sure that the silicone cushion clings tightly to the breast, otherwise the suction may be reduced.

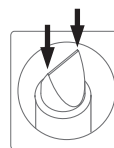**Breast milk buildup in the breast shield body:**

- If breast milk doesn't appear to be draining fast enough into the collection bottle, reduce the suction setting on your Electronic Breast Pump to allow the milk to drain before gradually increasing.
- Make sure that you keep breast pump properly. Bottle should be placed in an upright position.

HOW TO STORE AND USE YOUR EXPRESSED BREAST MILK?

- Milk can be stored in bottle or in breast milk storage bags.
- Freeze only fresh milk. Do not fill the bottle completely as frozen breast milk increases its volume.
- Breast milk can be heated in the same way as any other feed (but never in the microwave as this may destroy valuable nutrients and antibodies).
- Frozen breast milk can be defrosted overnight in the refrigerator, the warmed gradually in bottle warmer or hot water.
- Shake the bottle to combine separated fat molecules with the remaining volume of milk.
- Once defrosted, milk should be served to your baby within 24 hours. It should be stored in the fridge at this time.
- NEVER refreeze defrosted milk.
- DO NOT add fresh breast milk to already frozen milk for storage.
- DO NOT add fresh breast milk to already frozen milk.
- NEVER heat breast milk by boiling or putting it in the microwave.
- ALWAYS check food temperature before feeding.
- Breast milk that was not used should be disposed.

	Fresh breast milk	Defrosted breast milk
Room temperature	Up to 12h	Do not store
Refrigerator (+4°C)	2-5 days	24h
Freezer (-10°C)	2 weeks	Never refreeze defrosted milk
Deep freezing (-18°C do -20°C)	Up to 6 months	Never refreeze defrosted milk

As the Breast Pump is for personal use, it cannot be refunded. In the case of mechanical failure, please contact your local distributor or manufacturer. Please, keep your receipt as a proof of purchase. The warranty does not cover the parts of the device undergoing natural wear and tear as a result of normal use. The warranty also does not cover the faults resulting from the influence of external atmospheric factors (e.g. moisture, high or low temperatures).

WARNING!

- Do not use this product for purposes other than those listed in the attached instruction.
- Do not place the product near sources of fire.
- Before using the pump, make sure that all parts were assembled correctly.
- Immediately stop using the breast pump if you feel breast or nipple discomfort.
- Wash your hands before using the product.
- Keep the product out of reach of children.
- Do not drop the product to avoid any damage.
- To avoid burns do not touch the product with bare hands immediately after sterilization in boiling water.

MANUFACTURER DECLARATION

The manufacturer declares with full responsibility that the product complies with the essential requirements of the New Approach Directives, leading to CE marking. For confirmation the declaration of conformity is issued.

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